

FC-Life

Apnea Monitor

Superior Technology for Veterinary Monitoring





Advanced Performance & Ease of Use

Easy to use, simple and practical anesthetic Apnea Monitor to monitor each breath during anesthesia.

Just connect the Apnea Monitor between the endotracheal tube and your anesthesia machine and you are set to go.

Breathe Safe's sophisticated analysis algorithm will recognize and beep to indicate a breathe from a 1 pound kitten or a 200 pound Great Dane.

The monitor will alert if the patient fails to breathe for 30 seconds.

Features automatic turn-on and shut-off, strong and durable!



Features

Easy to Use: Only connect the Pet Apnea Monitor between the endotracheal tube and your anesthesia machine, then you are set to go!

Sensitive to Apnea: If your pet fails to breathe for 15 seconds, Pet Apnea Monitor will start to sound beep.

To Turn on Automatically: Just connect with the pet. The first breath of your pet will turn on the Pet Apnea Monitor automatically!

To Turn off Automatically: When the pet is disconnected, the Pet Apnea Monitor will wait 25 seconds, then beep for 20 seconds before turning off.

Built-in Battery: Pet Apnea Monitor adopts a long-life Lithium battery to provide more than 1 year of service. No battery changes or maintenance.

Easy to Clean: Use a cloth dampened with isopropyl alcohol 70% to wipe it, then use a clean cloth to dry it before use. Do not immerse it in liquid or sterilize it with autoclave.

Important: When you don't use it, please keep the Apnea Monitor in plastic package (Plastic Ziplock Bags) or a drawer to avoid triggering its sensitive function.

One year manufacturer's warranty for manufacturing defects.

